



shijiazhuang tiantian technology co.,ltd  
hadermal-filler.com



## Prophilo H+L 32mg 1-2ml Prefilled Syringe High Molecular Weight Hyaluronic Acid

Our Product Introduction

for more products please visit us on [hadermal-filler.com](http://hadermal-filler.com)

### Basic Information

- Place of Origin: Korea
- Brand Name: proffhilo
- Certification: CE
- Model Number: 2ml
- Minimum Order Quantity: 1box
- Price: 99USD
- Packaging Details: box
- Delivery Time: 5-8 work days
- Payment Terms: T/T, Western Union, MoneyGram paypal ,credit card
- Supply Ability: 1000box/month



### Product Specification

- Product Name: Prophilo
- Packing Size: 1 X 2ml Syringe
- Materials: Hyaluronic Acid
- Content: 32mg 64mg
- Needle: 29g



### More Images



## Product Description

Product Description :

Profhilo® H+L is a stabilized hyaluronic acid (HA) based product intended for skin remodeling and treating skin laxity, not just for filling lines and wrinkles. Profhilo® H+L is to be used in areas affected by skin laxity such as the malar and submalar areas of the face. Profhilo® H+L nourishes the dermal cells and restores the firmness of the skin. Profhilo® H+L improves tissue quality significantly, even in challenging areas, and is complementary to other aesthetic treatments and dermal filler procedures.

Specification :

Product name	Profhilo
content	2ml
materials	32mg hyaluronic acid

What are the most common question we get around this HA Gel?

### What is Profhilo

Profhilo is a **stabilized hyaluronic acid (HA) based gel** which is intended for skin remodeling and treating skin laxity, not just for filling facial lines and wrinkles. It nourishes the dermal cells and restores the firmness of the skin when it ages, as well as improves tissue quality significantly.

### Why is Profhilo unique?

There are many characteristics that make Profhilo unique from anything else available on the market right now. There is no BDDE content. It is thermally cross-linked generating stabilised hybrid co-operative complexes of high molecular weight hyaluronic acid and low molecular weight hyaluronic acid. Despite its high HA content, it has a low inflammatory profile coupled with pro-healing properties backed by clinic studies.

### How many mls of Profhilo is needed per treatment?

As a rule, for the entire procedure, **only one syringe of Profhilo is enough for one zone** Injections are made at 10 points on the face – 5 on one side and 5 on the other.

### How many clinic treatments with Profhilo are needed?

It is recommended to conduct an initial cycle consisting of **2 treatments performed with an interval of 30 days**. In the future, if necessary, maintenance treatment may be carried out every 2 months. However, it is recommended to develop an individual treatment plan with Profhilo depending on the severity of the patient's age characteristics.

*This dermal filler gel can be applied to different areas but the doctor will decide how, when to repeat the treatment and the after care.*

Of course, injection sites are not chosen randomly. Firstly, they must be anatomically safe, and secondly, targeted depending on the patient's needs. Injections can not be done at those points where there are exits of nerves, large blood vessels and lymph nodes. Otherwise, there is a risk of blocking blood flow or damage the nerve endings.

#### Format of this filler:

The format of this filler is **1 syringe of 2 ml (1x2ml)**

#### Recommended method of use:

Most of the times it is recommended to conduct an initial cycle consisting of 2 treatments performed with an interval of 30 days. In the future, if necessary, maintenance treatment is carried out every 2 months on the facial wrinkle or set of wrinkles.

However, it is recommended to develop an individual treatment plan with **Profhilo** depending on the severity of the patient's age characteristics, so results may vary or last differently

As a rule, for the entire procedure, **one syringe is enough for one zone, but this may vary** Injections are made at 10 points on the facial area – 5 on one side and 5 on the other.

# BENEFITS OF PROFHILO®

**Firming, tightening and lifting up the skin**

**Bio-stimulates elastin, 4 types of collagen and hydration in the 3 main layers of the skin**

**Repairing tissues, ideal for acne scarring**

**Improves overall skin condition and hydration**

**Fast and effective treatment with minimal pain and downtime**

NAHYCO  
HAEAL TECHNOLOGY

IBSA PROFHILO®

3.2% - 32 mg (H-HA) + 32 mg (L-HA)/2 ml  
Acido ialuronico sale sodico  
Siringa pre-empilata da 2 ml + 2 aghi

3.2% - 32 mg (H-HA) + 32 mg (L-HA)/2 ml  
Hyaluronic acid sodium salt  
2 ml pre-filled syringe + 2 needles



shijiazhuang tiantian technology co.,ltd



008619931162089



info@tiantianbike.com



hadermal-filler.com

NO 260 HUANAN ROAD SHIJIAZHUANG CITY HEBEI CHINA