

Filorga Hyaluronic Acid Meso Booster Nctf 135ha for Skin Rejuvenation

Our Product Introduction

Basic Information

- Brand Name: Filorga
- Certification: CE
- Model Number: 135HA
- Minimum Order Quantity: 1 box
- Price: 80 dollars
- Packaging Details: box
- Delivery Time: 1-3days
- Payment Terms: T/T, L/C, Western Union, MoneyGram
- Supply Ability: 5000box/month



Product Specification

- Product Name: Filorga 135HA
- Grade: 5x3ml Vials
- Type: 10x3ml Vials
- Ingredients: 0.025ml Cross-linked Hyaluronic Acid
- Keywords: NCTF 135HA
- 1st Phase —: 4 Treatments Every 7-15 Days.
- 2nd Phase —: 2 Treatments A Month Apart.
- 3rd Phase: 1 Treatment Every 2 – 6 Months.
- Stored: Between 2 And 25°C.
- What-s-App: +86-18132165209
- Highlight: **Filorga Hyaluronic Acid Meso Booster, Filorga 135HA skin rejuvenation injection, Filorga NCTF booster with hyaluronic acid**



More Images



for more products please visit us on hadermal-filler.com

Product Description

Most Popular France Filorga NCTF 135 Ha Mesotherapy Injections Anti-aging 5vials x3ml

Filorga 135HA 5mg/ml 5x3ml vials

WHAT IS FILORGA 135?

Manufactured by Laboratoires Filorga, this formula contains 55 unique active ingredients, each of which is used to treat photodamaged skin and reverse the signs of ageing. It actively repairs the skin, while also reducing deficiencies in minerals, including calcium, sodium, potassium, and magnesium.

Filorga 135 contains a hyaluronic acid concentration of 0.025mg/mL. Each package comes with 5 prefilled 3ml syringes, 5-18G withdrawal needles, 5-30G therapy needles, and 5-32G therapy needles. This fragile dermal filler should be stored between 2 and 25°C.

Specification :

Product name	135HA
Brand	Filorga
Country	Korea
Packing	5vials x3ml
Contains	hyaluronic acid concentration of 0.025mg/mL
Application	face, neck, décolletage, inside of the wrists, inner thighs, and abdomen
contains 1	55 active ingredients
Delivery terms	DHL,TNT,UPS,FEDEX
Delivery times	within 4-7days
Each package	10 prefilled 3ml syringes, 5-18G withdrawal needles, 5-30G therapy needles, and 5-32G therapy needles



WHAT IS IT USED FOR?

Filorga 135 was specifically formulated to enhance the hydration and flexibility of the skin, offering it a more radiant and healthy appearance. Its hyaluronic acid content replenishes lost endogenous hyaluronic acid, which redensifies the dermis, adds hydration to the skin, and promotes collagen regeneration.

This dermal filler may be used as a form of mesotherapy, working to treat the first signs of visible ageing in patients. These may include wrinkles and fine lines, elasticity loss, sagging, dullness, dehydration, and skin depressions.

This dermal filler leaves skin with a healthy glow, while smoothing out any rough texture and fine lines. Filorga 135 is intended for use on the face, neck, décolletage, inside of the wrists, inner thighs, and abdomen.



HOW DOES THIS DERMAL FILLER WORK?

After a single injection, Filorga 135's hyaluronic acid content helps the skin to retain moisture, leaving it with a healthy and refreshed appearance. Over time, the formula's other active ingredients work to stimulate the production of new skin cells, helping to improve the patient's overall skin quality.

The vitamins present in this facial filler work to stimulate cell functions, while its nucleic acid bases improve the communication between cells. Moreover, its coenzymes catalyse biochemical reactions, the amino acids stimulate protein production, and minerals prevent and repair skin deficiencies. Finally, its active antioxidants prevent free radicals from further damaging the skin. Together, these ingredients improve the appearance of the skin while also boosting its health over time.

HOW TO INJECT?

Only licensed, properly trained medical practitioners should perform treatments with Filorga 135. The treatment should be performed with an injector gun, or with the needles provided in the package.

This cosmetic filler may be injected into a variety of problem areas via the superficial dermis or the dermal-epidermal connective tissues. It is suitable for use in the abdomen and limbs, as well as the facial area, neck, and décolletage.

Please inform the patient that it will take a minimum of 5 treatment sessions, spaced over the course of 3 months, for them to see significant improvement.

HOW LONG DOES THIS FACE FILLER LAST?

Initial injections with Filorga 135 are to be performed over the course of 3 months, with maintenance sessions occurring every 3 months after this period.

When beginning treatment with this filler, the patient should be given 3 Filorga 135 injections with an interval of 15 days between each session. After this, the patient should receive 2 treatments spaced out by a month, followed by renewal sessions every 3 months.

However, due to the unique nature of each patient's skin, the physician should determine suitable intervals for treatment in each individual case. Please note that the revitalisation of mature skin will require more recurrent treatments than that of younger skin.

IS THIS PRODUCT SAFE?

Filorga 135 is a perfectly safe injectable solution for patient use, backed by 10 years of success and the treatment of more than a million patients. That being said, certain individuals should not be injected with this filler, including those who are sensitive to hyaluronic acid.

For a full list of the product contraindications, see the package insert.

SIDE EFFECTS:

While most patients react well to treatments with Filorga 135, some individuals may experience side effects following injection. However, these common reactions are usually mild in nature, and should resolve themselves within a week of initial treatment.

COMMON SIDE EFFECTS:

Swelling
Redness
Bruising
Itching
Nodules
Discolouration

LESS COMMON/RARE SIDE EFFECTS:

Necrosis

Abscess
Granuloma
Hypersensitivity

WHAT IS FILORGA 135 MADE OF?

Filorga 135 a hyaluronic acid-based filler, allowing it to add volume and hydration to the skin through its unique water retaining properties. With this, it plumps up ageing, creased skin, working to smooth out its wrinkles and fine lines. Additionally, the hyaluronic acid in this filler works to stimulate the production of elastin and collagen, improving the skin's overall texture and appearance.
In addition to hyaluronic acid, Filorga 135 contains 55 active ingredients.



shijiazhuang tiantian technology co.,ltd



008619931162089



info@tiantianbike.com



hadermal-filler.com

NO 260 HUANAN ROAD SHIJIAZHUANG CITY HEBEI CHINA